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One-touch
alergo™
Phototherapy Treatment

CLINICALLY APPROVED

FULLY AUTOMATIC

**DEVICE FOR THE TREATMENT OF RHINITIS ALLERGIC
USING PHOTOTHERAPY**
MODEL: VM-911

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CONTENTS

SAFETY INFORMATION	15
INTRODUCTION	16
ALLERGIC RHINITIS	17
OPERATING PRINCIPLES	18
INDICATIONS	19
DESCRIPTION	20
USING THE ALERGO VM-911	21
MAINTENANCE	21
INSTALLING THE BATTERIES	22
TECHNICAL SPECIFICATIONS	23
WARRANTY	36

Dear User,

Thank you for purchasing this device for the treatment of allergic rhinitis using phototherapy, the Alergo VM-911 by VISIOMED. We wish you a successful treatment and **we recommend that you read this manual carefully to make yourself familiar with the product features.**

THE MANUFACTURER RESERVES THE RIGHT TO MODIFY THE TECHNICAL CHARACTERISTICS OF THE PRODUCT WITHOUT PRIOR NOTICE.
--

SAFETY INFORMATION

For your **safety**, it is recommended that you read this manual carefully and adhere to the following instructions:

- Follow the maintenance instructions listed in this manual
- Follow the dosage indications
- This device is designed solely for use in a familial context
- Use this device only as intended and described in this manual; any other use will result in the warranty being void
- Do not use this device where relative humidity is greater than 80%
- Do not use the device whilst driving or operating machinery
- Do not use the device whilst showering
- This device must be kept in a clean and dry place
- Keep away from children
- Adult supervision is strongly advised whilst this device is being used by persons with restricted physical, sensory or mental abilities (including children)
- Shock may damage this device
- Only use the accessories and parts specified by the manufacturer or sold with the device
- Do not dismantle the device; maintenance and repairs should be carried out by a qualified technician
- Never submerge the device in water or any other liquid
- Do not use the device if any part has become damaged, submerged in water, or has been dropped; immediately return the device to the customer services department for it to be checked and repaired

- Do not use any other battery than those listed, do not recharge non-rechargeable batteries, do not throw batteries onto a fire
- Remove the batteries if the device will be unused for a long period
- Adhere to the correct polarity when inserting the batteries; inserting batteries incorrectly may damage the device and void the warranty
- Respect local regulations regarding the disposal of batteries
- Respect environmental protection rules when disposing of unwanted packaging

IF YOU ARE IN ANY DOUBT AS TO THE USE OF THIS PRODUCT,
PLEASE ASK YOUR DOCTOR.

INTRODUCTION

The Alergo VM-911 is a device for the treatment of allergic rhinitis by phototherapy.

Quick and simple to use (3 minutes per treatment), and completely pain-free, the Alergo VM-911 can be used at any time and anywhere.

The red light emitted by the Alergo VM-911 has no secondary effects, or harmful consequences for the nasal mucous. There are no known interactions with other medicines. The therapy acts at a localised level and has no effect on other parts of the body.

Treatment may be conducted over a prolonged period, a maximum of 3 sessions per day.

This device complies with Directive 93/42/EEC regarding medical devices. This device complies with safety standards ISO-13485, EMC, ISO-10993, and EN60601-1.

ALLERGIC RHINITIS

Allergic rhinitis is defined by WHO (World Health Organisation) as an acute or chronic inflammation of the nasal mucous (internal covering of the nose), caused by allergens.

When an allergen is absorbed, the human body produces an allergic reaction and releases a substance, histamine, which is mainly responsible for the typical allergic symptoms.

This reaction is clinically characterised by at least one of the following symptoms:

- Sneezing,
- Rhinorrhea (runny nose)
- Nasal congestion (blocked nose)
- Nasal pruritus (itching in the nose)
- Inflammation of the nasal mucous

The symptoms are proportional to the concentration of allergens.

Other symptoms may also present:

- Headaches (pain in the head),
- Sensitive face,
- Conjunctivitis,
- Fatigue,
- Loss of concentration and energy
- Etc.

There are many types of allergic rhinitis:

1. **Seasonal allergic rhinitis**, more commonly known as hay fever, caused pollen from trees, grasses and various herbaceous plants. It occurs each year at the same season (see the pollen calendar to ascertain periods when the pollen count is high), and is often associated with conjunctivitis.

2. **Aperiodic allergic rhinitis**, also known as chronic “perannual” allergic rhinitis, often associated with asthma. Symptoms occur all year round, particularly in winter (people spend long periods of time indoors).

The allergens associated with this type of allergic rhinitis are always present in our surroundings, for example, dust, animal hair, or mites.

3. **Chronic non-allergic rhinitis**, or “vasomotor rhinitis”, is persistent and presents due to environmental factors, such as temperature variation, humidity, stress, atmospheric pollution, tobacco smoke, alcohol etc...

4. **Work-related rhinitis**, caused by exposure to substances in the place of work.

OPERATING PRINCIPLES

Phototherapy is a new form of treatment for allergic rhinitis, without medicine.

This technique uses two non-heating red lights to treat allergic reactions in the nasal mucous. The rays of light generated by the diodes in the Alergo use two wavelengths simultaneously:

- 652 nanometres (red light): this wavelength eliminates allergic symptoms and reduces hypersensitivity in nasal mucous.
- 940 nanometres (not visible to the naked eye): this wavelength helps to stimulate blood circulation.

Phototherapy has a strong immunosuppressant effect (eliminating allergic symptoms) and is able to inhibit hypersensitivity reactions in the nasal mucous.

INDICATIONS

The Alergo VM-911 may be used either preventively or as a treatment during the symptoms manifestation.

Do not exceed a maximum of 3 sessions per day.

PREVENTIVE TREATMENT:

According to the period during which your allergen occurs (see pollen calendar), begin the treatment one week before symptoms would normally appear.

1 session of 3 minutes per day, preferably in the morning.

TREATMENT:

3 sessions of 3 minutes per day; morning, midday and evening.

It is advisable to continue the treatment for 2 to 3 weeks.

See the pollen calendar for more detail on allergic periods.

Clinically tested results:

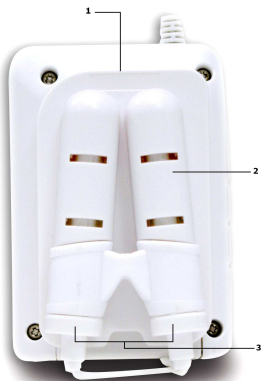
Clinical tests of the Alergo VM-911 have shown a significant reduction in the classic symptoms of allergic rhinitis. Following treatment, a characteristic improvement in the general state of patients was noted.

DESCRIPTION


1. Battery compartment
2. Nasal Tube compartment
3. Nasal clamp with nasal tubes

1. "One Touch" button – On/Off
2. Nasal clamp with nasal tubes
3. Light emitting diodes – wavelength 652 nanometres (red light visible to the naked eye) and 940 nanometres (not visible to the naked eye)

USE

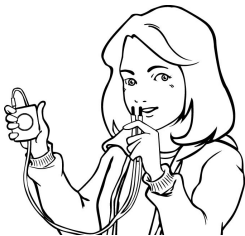- ➊ Position the nasal tubes comfortably in the nostrils.
- ➋ Press the “One Touch” button; the diodes will turn on, the Alergo will beep 2 times and the treatment begins.
- ➌ The light from the Alergo VM-911 turns off automatically after 3 minutes of treatment. The Alergo will beep 8 times. The session is finished.
- ➍ Remove the nasal tubes.

You may also press the “One Touch” button to turn off the device during the course of a session.

MAINTENANCE

1. Use a cloth damped with 70% concentration alcohol to disinfect the nasal tubes after each use.
2. Do not use alcohol, benzene, solvent or any other chemical product to clean the rest of the device.
3. Avoid placing the unit in direct sunlight, areas of high humidity or dusty areas.
4. Never submerge the device in water or any other liquid.
5. If the device will be unused for a prolonged period, remove the battery from the device to avoid any leakage which may damage the product.

CHANGING THE BATTERIES

The Alergo VM-911 must only be used with a 9 volt heavy duty dry battery.

If the light in the “One Touch” button are flashing, the battery is too low. Replace the battery to avoid affecting treatment.

If the light in the “One Touch” does not turn on, the battery has not been inserted correctly or is empty.

1. Open the battery compartment.
2. Replace the used battery, taking care to adhere to the correct polarity. Inserting batteries incorrectly may damage the device and void the warranty.
Never insert rechargeable batteries. Use single-use batteries. Do not mix new and old batteries.
3. Close the cover.

Do not dispose of used batteries in household waste.
Take used batteries to a collection point.

If the device will be unused for a prolonged period, remove the battery from the device to avoid any leakage which may damage the product.

TECHNICAL SPECIFICATIONS

Model No.:	VM-911
Product name:	Alergo
Indications:	Double wavelength phototherapy for rhinitis allergic reactions in nasal mucous.
Position of diodes:	Nostrils
Wavelength:	652 nanometres 940 nanometres
Light emitted:	Continuous red light
Energy consumption:	3mw per diode
Unit of energy per treatment:	1.0 joule
Treatment duration:	Approximately 3 minutes
Usage conditions:	15°C ~ 40°C (59°F ~ 104°F)/ 30% to 80% relative humidity
Storage conditions:	-10°C ~ 60°C (14°F ~ 140°F)/ 30% to 80% relative humidity
Power supply:	1 x 9V heavy duty dry battery
Battery life:	Approximately 100 sessions (approximately 3 minutes per treatment)
Dimensions:	4.7 x 4.5 x 7.8 cm
Weight:	87 grams
Accessories:	User manual, 1 x 9V battery
THE MANUFACTURER RESERVES THE RIGHT TO MODIFY THE TECHNICAL CHARACTERISTICS OF THE PRODUCT WITHOUT PRIOR NOTICE.	

One-touch
alergo™
Phototherapy Treatment

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This product carries two years guarantee for parts and labour. In the event of any problem, please contact your retailer.

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Note: The guarantee will not apply if the device is disassembled or used improperly. Please keep your guarantee card.

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Service Technique
0 892 350 366
(0.34 €/min)

English

Disposal of Waste Equipment by Users in Private Households in the European Union

The symbol on the product or on its packaging indicates that this product must not be disposed of with your other household waste. Instead, it is your responsibility to dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.

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VISIOMED SAS FRANCE
Visiomed SAS France
Bannex Ltd