FR

PATCH CHAUFFANT

KINECARE®

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VISIOMED GROUP SA  
112, Avenue Kléber  
75116 PARIS - France

DISTRIBUÉ PAR / DISTRIBUTED BY

Marché Européen /  
European market

VISIOMED GROUP SA  
112, Avenue Kléber  
75116 PARIS - France  
contact@visiomed-lab.fr  
Tel : +33 (8) 92 350 334  
(0,40€/min)  
www.visiomed-lab.com

BAS  
DU DOS

BAS DU DOS  
10 X 13 CM

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LOWER  
BACK

LOWER BACK  
10 X 13 CM

## Medical device class IIa, compliant with European Standards.

### CONTENT

1. COMPOSITION.....	6
2. SAFETY NOTES.....	6
2. 1. WARNINGS.....	6
2.2. PRECAUTIONS.....	7
3. INDICATIONS.....	8
4. USE.....	8
5. CONSERVATION.....	8

Dear customer,

You have just purchased a KINECARE product. In order to use it optimally and efficiently, we recommend that you read these operating instructions carefully.

### 1. COMPOSITION

Iron, water, activated carbon, salt.

### 2. SAFETY NOTES

Only use this device for its intended purpose as described in this notice.

#### 2. 1. WARNINGS

- This product is for single use only.
- This product is intended for external use only.
- Do not use after the stated expiry date.
- Do not put in direct contact with the skin without the use of belt. Risk of burns.
- This device is suitable for home use.
- Keep this product in its original packaging in a clean, dry place. Store the product at room temperature 4-30°C (40-86°F). Do not freeze it.
- Each heating cell contains iron (about 2 g). Do not put in the mouth. If swallowed, immediately contact a doctor.
- Do not get in eyes. In case of contact with eyes, immediately flush with clean water and contact your doctor.
- Keep out of reach of children and pets.
- The use of this device is not intended for persons (including children) with limited physical, sensory or mental abilities or lack of experience and/or lack of knowledge, unless they are supervised by a person responsible for their safety or have been instructed on the use of the device.

- Do not use on persons who cannot remove the patches themselves, without external assistance, or are unable to follow precautions for use, including children and the elderly.
- Never heat or re-heat the product in a microwave oven - risk of fire.
- Do not use on children under the age of 12.
- This product should not encourage self-medication or the adaptation of your treatment. Do not use the product if the protective pouch or patch is damaged (leakage, deterioration, tearing).
- Do not use if the protective film is damaged or missing.

#### 2.2. PRECAUTIONS

- Do not apply more than 12 hours per day on the same area.
- Do not use if bruising or swelling of less than 48 hours.
- Do not use on parts of the body that are insensitive to heat.
- Do not put in direct contact with the skin without the use of a belt / knee pad. Risk of burns. Do not use on injured skin.
- Do not sleep with the product.
- **Consult your physician before use:** In case of diabetes, circulatory disorders, cardiovascular diseases, rheumatoid arthritis, skin disorders, pregnancy, skin insensitivity (Raynaud's syndrome). If you have any doubts about the use of this heating patch, ask your doctor for advice.
- **If you are 55 or older**, the risk of burns is increased, so it is recommended that you wear the Kinicare patch over a garment and not use it when sleeping.
- Do not use if you have ever had an allergic reaction to a similar patch.
- This product may cause blisters or skin burns. If the product appears too hot, stop using it or wear the patch over a garment. Check your skin frequently. Immediately remove the patch in case of discomfort, irritation or burning.
- Consult a physician if you experience discomfort, burning, swelling, rash, or other persistent changes in your skin where the patch has been placed.
- Ask a doctor if the pain worsens or persists for more than 7 days.
- Do not use on face, head, chest, inner arm, elbow creases, armpits, fingers, groin, knee pits or feet.
- To avoid increasing the risk of burns, remove the product before undergoing any MRI (magnetic resonance imaging) examination.
- Do not use with local analgesic products (such as creams, balms, lotions, liniments, etc.).
- Do not use in combination with other medicines for local use or other sources of heat.
- Do not apply additional pressure, do not lean against hard surfaces, do not wear bandage or tight clothing or a belt over the patch.

### 3. INDICATIONS

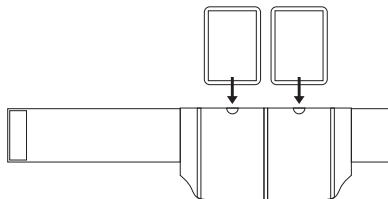
KINECARE patches are composed of calorific agents (iron, activated carbon, water...) that produce heat in contact with oxygen in the air. They act quickly and provide a natural treatment, which allows the release of 12h of constant and soothing heat.

Relieves in case of Low back pains: Back-spinal pain, lumbagos, Sciatica, blows, hematomas.

### 4. USE

**Only open the protective pouch at the time of application.**

1. Remove the patch from the protective sleeve.
2. Insert the heated patches into the appropriate location on the belt pad.

3. Be sure to well position the belt pad with the velcro on the treated area.
4. In contact with air, the patches heats up gradually. Approximately 30 min. is needed to reach the therapeutic temperature.
5. The patches can be worn continuously for 12 hours. However, it is advisable to check the good tolerance of the heat beyond 8h of use.
6. Remove the patches by gently lifting the ends.
7. This product is for single use only. Discard after use.

### 5. CONSERVATION

This product is for single use only.

Store below 25°C.

Waste disposal: This product is not a hazardous waste.

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© 07/2017 VM-PC4
**EN:** Kinecare® is a brand registered by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.