ZAP X™

ZAP X™

## SAFETY PRECAUTIONS

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USER MANUAL

## TABLE OF CONTENT

<b>I. SAFETY INSTRUCTIONS</b>	<b>15</b>
<b>II. INTRODUCTION</b>	<b>15</b>
<b>III. PRECAUTIONS BEFORE USE</b>	<b>16</b>
<b>IV. PRINCIPLE OF OPERATION</b>	<b>17</b>
<b>V. STARTING UP YOUR ZAP'X VM-X100</b>	<b>17</b>
<b>V. 1. DESCRIPTION OF THE APPLIANCE AND THE SUPPLIED ACCESSORIES</b>	<b>17</b>
<b>V. 2. INSTALLATION OF THE BATTERIES</b>	<b>18</b>
<b>V. 3. RECOMMENDATIONS FOR EFFECTIVE USE</b>	<b>18</b>
<b>VI. USE OF THE ZAP'X™</b>	<b>18</b>
<b>VII. SERVICING THE APPLIANCE</b>	<b>19</b>
<b>VIII. TROUBLESHOOTING GUIDE</b>	<b>20</b>
<b>IX. TECHNICAL SPECIFICATIONS</b>	<b>20</b>

The manufacturer reserves the right to change the technical  
Characteristics of the product without prior notice.

Dear customer,

Thank you for having purchased this Visiomed® head lice comb. We wish you excellent use of it and recommend that you read this User Guide very carefully, in order to familiarise yourself with the characteristics of the ZAP'X™ VM-X100.

## I. SAFETY INSTRUCTIONS

For your safety, it is recommended that you read this User Guide very carefully, and comply with the following instructions:

- Follow the servicing instructions provided in this guide.
- This appliance can be used either professionally or individually at home.
- Use this appliance only for the purpose for which it was designed and as described in this guide.
- Do not use this appliance in relative humidity greater than 85%.
- This appliance must always be kept in a clean, dry place.
- Do not leave this appliance within reach of children.
- A shock or impact can damage this appliance.
- Use only the accessories and spare parts specified by the manufacturer or sold with the appliance.
- Do not dismantle the appliance. Entrust the servicing of this appliance to a qualified technician.
- Do not operate the appliance close to a mobile telephone, a microwave oven, or close to a television set (within less than one metre).
- Do not expose the appliance to direct sunlight.
- Never immerse the appliance in water or any liquid whatever.
- Do not use a battery other than the batteries mentioned, do not re-charge non-rechargeable (dry) batteries, and do not dispose of them in a fire. Remove the batteries if the appliance is not to be used for a long period.
- Observe the polarity of the batteries. A polarity error can damage the appliance and compromise the guarantee of your appliance.
- Comply with local regulations relating to the treatment of waste when disposing of used batteries.

## II. INTRODUCTION

Combating head lice in children is a veritable war when an infection is raging. The traditional ritual of treatments and repellents are effective in general, but are restrictive, not only for both the child and the parents, but also for the whole family circle. Medical treatments, and special anti-parasite creams, shampoos and lotions allow the parasite to be eradicated, but it can also happen that some of these treatments lose their effectiveness, and one then has to make several attempts before achieve a result.

### ZAP'X™ is the alternative to medical treatments

The ZAP'X™ VM-X100, an anti-lice comb developed by the Visiomed® laboratory, is the effective alternative to medical treatments that are sometimes noxious for the child after several attempts. It has been designed specially in order to combat an invasion of lice and parasites in man, in a simple and effective way, and with no danger to the hair or the scalp.

The ZAP'X™ VM-X100 is totally painless and inoffensive, even for very small children. It is easy and quick to use. In contact with insects, the teeth of the ZAP'X™ generate an electrical voltage of 100 Volts, which is imperceptible to the human but kills the lice on the spot by electrocution.

Moreover, when brushing, the toothed structure, in extruded steel, loosens the nits and removes them from the hair at the same time as the lice. Since there is no active substance involved in the use of the head lice comb, you are able to repeat brushing with the ZAP'X™ VM-X100 process as often as necessary.

### III. PRECAUTIONS BEFORE USE

For optimal use of the ZAP'X™ head lice comb, it is recommended that you comply with these instructions and obey the following guidelines:

- Use of the ZAP'X™ VM-X100 is not recommended in the following cases:
  - Wearers of a cardiac stimulation device.
  - People suffering from epilepsy or heart problems.
- Do not use on damp or wet hair.
- Do not leave the appliance within the reach of children.
- Never use the appliance on another member of the family without first disinfecting the head lice comb.
- Disinfect the appliance and the teeth of the comb after each use, by means of a cloth soaked in a 70% alcohol solution (be sure to remove power from the appliance when cleaning it, and wait until it is totally dry before using it again).
- Do not use the comb if the teeth of the latter are misshapen, since this may cause the appliance to short circuit.
- Use this appliance only for the purpose for which it was designed, and as described in this guide.

### IV. PRINCIPLE OF OPERATION

The ZAP'X™ X100 is an electronic head lice comb generating a voltage of 100 volts (powered by one AAA battery).

The integrated circuit of the comb generates a periodic square-wave oscillation when the louse comes into contact with the teeth of the ZAP'X™.

The electric shock is triggered automatically; the lice are destroyed instantaneously.

The low value of the current generated by the 1.5 V battery means that the ZAP'X™ VM-X100 is totally harmless and without danger for the human organism.

### V. STARTING UP YOUR ZAP'X VM-X100

#### V. 1. DESCRIPTION OF THE APPLIANCE AND THE SUPPLIED ACCESSORIES

- 1 On/Off switch
- 2 Comb
- 3 Battery compartment
- 4 Lid of the battery compartment
- 5 Cleaning brush

## V. 2. INSTALLATION OF THE BATTERIES

- Open the lid protecting the battery.
- Insert the supplied AAA battery, taking care to observe the correct polarity. A polarity error can damage your appliance and compromise the guarantee. Never insert rechargeable batteries. Employ only single-use batteries. Do not mix new batteries with used batteries.
- Replace the lid.

Do not dispose of your used batteries with your household waste. Take your used batteries to a collection centre. Where the appliance is not to be used for some time, be sure to remove the batteries from the appliance in order to prevent the leakage of liquids that could damage the unit.

## V. 3. RECOMMENDATIONS FOR EFFECTIVE USE

The ZAP'X™ head lice comb must be used on dry hair.  
Make sure that the appliance is not damp or wet during use.  
It is recommended that you use a repellent treatment (cream, shampoo, etc.) after use of the ZAP'X™, so as to prevent any fresh infection.

## VI. USE OF THE ZAP'X™

1. Press the On/Off switch upwards. A sound resembling a hum indicates that your comb is working and ready for use.
2. Hold the comb at an angle of 45° to the scalp. The black bulbs located at the ends of the teeth must be positioned facing the scalp. The lice are generally to be found in the warmer areas of the head, such as near the neck and below a ponytail.
3. Move the comb slowly through the hair, preferably lock by lock, for a more effective result. The direction of the brushing can from the bottom up or vice versa, taking care to maintain the 45° angle.
4. The humming sound stops briefly each time a louse is destroyed. If the hum does not resume after 5 seconds, use the small cleaning brush supplied with your appliance, in order to eliminate any lice caught in the teeth of the comb. During brushing, short-circuits due to contact between adjacent teeth can also lead to a silencing of the appliance.

5. Take the time to comb the hair in depth. Make sure that all the lice have been destroyed, and that the nits have been correctly released and removed.
6. You can use the ZAP'X™ head lice comb as often as necessary in any given day.
7. In order to prevent a return of the lice, it is recommended that you apply a repellent after using the head lice comb, especially if the child is mixing with others.
8. In confirmed periods of infection, frequent use of the ZAP'X™ is recommended, preferably at night before bedtime.

## VII. SERVICING THE APPLIANCE

### **Be sure to clean your head lice comb after every use.**

Never use the appliance on another member of the family without first having disinfected the head lice comb.

- Press the On/Off switch of the appliance in order to switch it off. To be extra careful, you can also remove the battery.
- Clean the appliance by gently disinfecting the surface of the appliance and the teeth of the comb, using a tissue soaked in ethyl alcohol (a 70% solution).
- The teeth of the comb represent a fragile part of the appliance. Take care not to damage them, as this can lead to short circuiting of your ZAP'X™
- Wait until the appliance is totally dry before using it again.

### VIII. TROUBLESHOOTING GUIDE

If any of the following problems occur during use of your head lice comb, please refer to this troubleshooting guide. If the problem persists, please contact our Customer Support service on +33 892 350 334.

<b>Problem encountered</b>	<b>Solution</b>
Nothing happens when you press the switch	Check the battery (polarity, state of charge)
The hum disappears or is intermittent during use	Check the state of charge of the battery, and change it if necessary. While brushing, short-circuits can occur due to contact between adjacent teeth, which can also lead to silencing of the appliance.

### IX. TECHNICAL SPECIFICATIONS

<b>Model</b>	ZAPX
<b>Reference</b>	VM-X100
<b>Class</b>	Medical device class I
<b>Frequency</b>	5.1KHz
<b>Tension</b>	98V/DC
<b>Frequency of the buzzer</b>	6.9KHz
<b>Indication</b>	Hum
<b>Power supply</b>	1 AAA battery
<b>Consumption</b>	≤40mA during use
<b>Work ambient temperature</b>	-10 - +40°C / 10 - 85%RH
<b>Storage conditions</b>	-10 - +50°C / 10 - 85%RH
<b>Quiescent Current</b>	<1 uA
<b>Dynamic Current</b>	<10 mA
<b>Voltage Range</b>	1.35 - 1.5V
<b>Power Consumption</b>	0.015W
<b>Battery life</b>	Six hours in continuous use
<b>Dimensions</b>	121 x 61.4 x 19.7mm
<b>Weight</b>	45g
<b>Standard</b>	CE

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**EN:** Visiomed<sup>d</sup> will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

**DURATION:** 12 MONTHS RETURN TO WORKSHOP

**LIMITS AND EXCLUSIONS:** This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user.

This guarantee covers only the defects of the material or parts, occurring during normal use of the product. It does not cover the damage caused during the transport of the apparatus, caused due to repairs being carried out by the distributor, by any modifications undertaken, any connection of equipment not approved by Visiomed<sup>d</sup>, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, floods, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal maintenance, cleaning and the replacement of parts where wear is normal, are not covered by the terms of this guarantee. Visiomed<sup>d</sup> and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Visiomed<sup>d</sup>, any other guarantee (commercial guarantee) except this one will not be taken into account.

**IMPORTANT:** During the guarantee period if you are dissatisfied with the repairs of this product, please contact the Visiomed<sup>d</sup> customer service.

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**EN:** Disposal of Waste Equipment by Users in Private Households in the European Union.

The symbol on the product or on its packaging indicates that this product must not be disposed of with your other household waste. Instead, it is your responsibility to dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.

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**EN:** ZAP'X is a brand registered by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.

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Compliant with the directive 93/42/EEC.

**PRECAUTIONS FOR USE**

- Keep out of children's reach
- Disinfect the ZAP™ after each use
- DIRECTIONS FOR USE**
- The ZAP™ VM-X200 is used on dry.
- Take a section of hair approximately 5 cm wide.
- Comb the hair section slowly from the root to the hair end.
- Renew this operation on all of the hair as often as necessary.

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**NL**

Conform de richtlijn 93/42/EEC. Medical device.

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**PT**

Conforme a directiva 93/42/EEC. Medical device.

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DISTRIBUÉ PAR / DISTRIBUTED BY

VISIONED GROUP SA  
112, Avenue Kléber  
75116 PARIS - France

Made in China  
**VISIONED Technology Co., LTD.**  
2<sup>nd</sup> Floor of N°1 building, Jia An  
Technological Industrial Park  
67 District, Bao An  
518101 SHENZHEN - China

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© 04/2017 VM-X100